70110160

## Quick mobile rail - small (40 cm)

English .....	<b>Instructions for use .....</b>	<b>P. 03</b>
Nederlands .....	<b>Gebruiksaanwijzing .....</b>	<b>P. 04</b>
Deutsch .....	<b>Gebrauchsanweisung .....</b>	<b>S. 05</b>
Français .....	<b>Mode d'emploi .....</b>	<b>P. 06</b>
Dansk .....	<b>Brugsanvisning .....</b>	<b>S. 07</b>
Eesti keel .....	<b>Kasutusjuhend .....</b>	<b>P. 08</b>
Español .....	<b>Manual de instrucciones .....</b>	<b>P. 09</b>
Italiano.....	<b>Manuale d'uso .....</b>	<b>P. 10</b>
Latviešu .....	<b>Lietošanas instrukcija .....</b>	<b>S. 11</b>
Lietuvių .....	<b>Naudojimo instrukcija .....</b>	<b>P. 12</b>
Norsk .....	<b>Bruksanvisning .....</b>	<b>S. 13</b>
Português .....	<b>Manual de instruções .....</b>	<b>P. 14</b>
Русский .....	<b>Инструкция по применению .....</b>	<b>P. 15</b>
Slovenščina.....	<b>Navodila za uporabo .....</b>	<b>P. 16</b>
Suomi .....	<b>Käyttöohje .....</b>	<b>S. 17</b>
Svenska .....	<b>Bruksanvisning .....</b>	<b>S. 18</b>

# VITALITY

# VITILITY

## ENGLISH INSTRUCTIONS FOR USE

# Quick mobile rail - small (40 cm)

## 1. INTENDED USE

The Quick mobile rail is designed to provide support, for example when getting in and out of the shower. The Quick mobile rail can be attached and removed within a few seconds with suction cups. Drilling holes or the use of screws is not necessary. The two powerful suction cups are pressed firmly onto a non-porous, sturdy surface. Just press the vacuum handles down to fix the Quick Mobile rail.

## 2. RISKS & WARNINGS

The evaluation of all risks by VITILITY risk management showed that the remaining residual risk is as low as reasonably possible.

### 2.1 SAFETY GUIDELINES

Please ensure the product is in a faultless condition before use.

### 2.2 POTENTIAL RISKS

#### - INCORRECT ASSEMBLY OR INSTALLATION

Incorrect assembly or installation can lead to reduced performance or dangerous situations.

#### - INSUFFICIENT SUPPORT

The product may provide insufficient support, which can cause discomfort, injury, or worsen physical limitations.

#### - INSTABILITY

Insufficient sturdiness or stability of the product can lead to tipping or falling, which can cause injury or damage.

#### - OVERLOADING DUE TO INCORRECT USE

Using the product in a way it was not designed for can lead to damage to material or structure.

#### - USE BY YOUNG CHILDREN

This product is not suitable for children under 6 years. Use is only allowed under strict adult supervision to prevent possible dangers or misuse.

### 2.3 WARNINGS

The product must be replaced if required by one or more of the following criteria:

- Do not use if the product is damaged.
- The user's general condition is an important part of the risk assessment.

### 2.4 PRECAUTIONS

- People with self-harming behavior should not be left alone with the product.

## 3. WARRANTY

There is a 12 months warranty on this product. The warranty period begins on the date of purchase (shown on

the purchase receipt). During the warranty period, you can submit any complaints about a defective VITILITY-product to your dealer or to the service address specified. In this period, we will repair all defects due to faulty materials or operation free of charge. Improper handling, falls, impacts and the like are excluded from the warranty. A new warranty period does not start if the product is repaired or exchanged; the 12 months warranty period as of the purchase date continues unchanged.

Any warranty expires in the event of non-compliance such as use other than as intended, repairs and alterations made to the head protector or its components by unauthorised people.

## 4. SYMBOLS

The symbols and their explanations can be found on page 19.

## 5. SERIOUS INCIDENT REPORT

In the event of serious incidents related to the use of the device should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward the information to the manufacturer.

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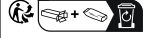
# VITILITY

## SYMBOLS

	Adjustable length (XX - XX cm)
	Adjustable width (XX - XX cm)
	Length (XX cm)
	Width (XX cm)
	Length x width (XX x XX cm)
	Length x width x depth (XX x XX x XX cm)
	Diameter (XX cm)
	Inner-diameter (XX cm)
	Adjustable angle (XX °)
	Maximal volume (XX L)
	Product weight (XX g)
	Maximal userweight (XX kg)
	Magnifies (X times)
	Suitable for dishwasher (till X °C)
	Unsuitable for dishwasher
	Suitable for microwave (till X °C)
	Unsuitable for microwave
	Handwash only

	Suitable for washingmachine (till X °C)
	Unsuitable for washingmachine
	Tools required
	No tools required
	One size
	Food safe
	Der grune punkt (The green dot)
	LOT - Identification number
	CE-Making
	Reference number
	Read manual before use
	Unique Device Identification
	Medical Device
	Stock keeping unit
	Manufacturer
	Recycle logo
	Forest Stewardship Council - logo
	Triman (France only)