VITALITY

70510160

Reacher - small

English	Instructions for use	P. 03
Nederlands	Gebruiksaanwijzing	P. 04
Deutsch	Gebrauchsanweisung	S. 05
Français	Mode d'emploi	P. 06
Dansk	Brugsanvisning	S. 07
Eesti keel	Kasutusjuhend	P. 08
Español	Manual de instrucciones	P. 09
Italiano.....	Manuale d'uso	P. 10
Latviešu	Lietošanas instrukcija	S. 11
Lietuvių	Naudojimo instrukcija	P. 12
Norsk	Bruksanvisning	S. 13
Português	Manual de instruções	P. 14
Русский	Инструкция по применению	P. 15
Slovenščina.....	Navodila za uporabo	P. 16
Suomi	Käyttöohje	S. 17
Svenska	Bruksanvisning	S. 18

VITILITY

VITILITY

ENGLISH INSTRUCTIONS FOR USE

Reacher - small

1. INTENDED USE

With the reacher it is possible to pick up items which are situated too high or too low, for example in a closet or on the floor, without having to bend or stretch. By squeezing the handle lightly, the reacher closes around the object. The reacher is suitable for holding small and lightweight items, such as remote controls, magazines, keys, etc.

2. RISKS & WARNINGS

The evaluation of all risks by VITILITY risk management showed that the remaining residual risk is as low as reasonably possible.

2.1 SAFETY GUIDELINES

Please ensure the product is in a faultless condition before use.

2.2 POTENTIAL RISKS

- CUT OR PINCH HAZARD

There is a risk of cuts or pinching injuries when contacting moving or sharp parts.

- USE BY YOUNG CHILDREN

This product is not suitable for children under 6 years. Use is only allowed under strict adult supervision to prevent possible dangers or misuse.

2.3 WARNINGS

The product must be replaced if required by one or more of the following criteria:

- Do not use if the product is damaged.
- The user's general condition is an important part of the risk assessment.

2.4 PRECAUTIONS

- People with self-harming behavior should not be left alone with the product.

3. WARRANTY

There is a 12 months warranty on this product. The warranty period begins on the date of purchase (shown on the purchase receipt). During the warranty period, you can submit any complaints about a defective VITILITY-product to your dealer or to the service address specified. In this period, we will repair all defects due to faulty materials or operation free of charge. Improper handling, falls, impacts and the like are excluded from the warranty. A new warranty period does not start if the product is repaired or exchanged; the 12 months warranty period as of the purchase date continues unchanged.

Any warranty expires in the event of non-compliance such as use other than as intended, repairs and alterations made to the head protector or its components by unauthorised people.

4. SYMBOLS

The symbols and their explanations can be found on page 19.

5. SERIOUS INCIDENT REPORT

In the event of serious incidents related to the use of the device should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward the information to the manufacturer.

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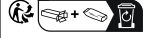
VITILITY

SYMBOLS

	Adjustable length (XX - XX cm)
	Adjustable width (XX - XX cm)
	Length (XX cm)
	Width (XX cm)
	Length x width (XX x XX cm)
	Length x width x depth (XX x XX x XX cm)
	Diameter (XX cm)
	Inner-diameter (XX cm)
	Adjustable angle (XX °)
	Maximal volume (XX L)
	Product weight (XX g)
	Maximal userweight (XX kg)
	Magnifies (X times)
	Suitable for dishwasher (till X °C)
	Unsuitable for dishwasher
	Suitable for microwave (till X °C)
	Unsuitable for microwave
	Handwash only

	Suitable for washingmachine (till X °C)
	Unsuitable for washingmachine
	Tools required
	No tools required
	One size
	Food safe
	Der grune punkt (The green dot)
	LOT - Identification number
	CE-Making
	Reference number
	Read manual before use
	Unique Device Identification
	Medical Device
	Stock keeping unit
	Manufacturer
	Recycle logo
	Forest Stewardship Council - logo
	Triman (France only)