Arm sling


DE: Armschlinge
DK: Armslynge
EE: Lingside käsivarrelle
EN: Arm sling
ES: Cabestrillo
FI: Käden tukiside

FR: Écharpe médicale
IT: Reggibraccio
LT: Rankos įtvarys
LV: Rokas nofiksēšanas ortoze
NL: Mitella
NO: Armslynge

PT: Suporte de braço
RU: Перевязь для руки
SE: Mitella
SI: Opornica za roko

22 cm
8.7 inch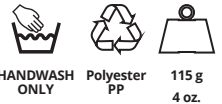

HANDWASH
ONLY

Polyester
PP

115 g
4 oz.

English (EN):

Thank you for choosing our VITILITY product. We aim to ensure that you fully enjoy your experience with our product. This manual is your comprehensive guide to understanding and using our product to its full potential. Keep it for future reference. In case it gets lost, you can always find a copy on our website at www.vitality.com.

Be sure to read all information and directions before use! Keep this manual for future use.

Intended use

The VITILITY Arm sling is a medical device designed to support the arm, wrist, or shoulder after injury or surgery. It provides stability and helps to reduce strain during recovery.

The sling is intended for use in home care settings and medical environments such as hospitals, rehabilitation centres, and physiotherapy clinics.

The adjustable strap ensures a custom fit for comfort and effective immobilization.

This product conforms to the requirements of the Medical Devices Regulation (EU) 2017/745. This manual applies to the VITILITY Arm sling – 70610270 (2025).

Contra-Indications:

- Circulatory Issues: Individuals with impaired blood circulation in the

arm should consult a healthcare professional before use.

- Severe Shoulder Dislocation: Use only under medical supervision.
- Cognitive Impairments: Patients who cannot understand or comply with proper use should be monitored during use.

Warnings:

- Correct Fit: Always adjust the strap to ensure the sling supports the arm evenly.
- Supervision: Use under medical supervision after surgery or injury.
- Discontinue Use: Stop using the product if you experience numbness, swelling, or increased pain.
- Inspection: Do not use the sling if it is damaged or worn out.

Maintenance:

The sling can be washed by handwash only using mild detergent.

Do not bleach.

Allow to air dry completely before reuse.

Do not iron or tumble dry.

Regularly inspect the product for signs of wear and tear, such as frayed straps or stitching.

This product does not require further maintenance.

Warranty, service life

We provide a 1-year warranty covering material and manufacturing defects. For detailed terms and conditions, please visit www.vitality.com.

- The product has a service life of 2 years, depending on frequency of use and proper maintenance.
- If the product has been in storage for an extended period, inspect the condition before use.

Storage

Store in a cool, dry place, away from direct sunlight.

Materials

Sling Body: Polyester

Buckle: PP

Use:

To use the arm sling:

- Place the injured arm into the sling so the elbow rests securely in the corner.
- Bring the strap over the opposite shoulder and secure it using the adjustable buckle.
- Adjust the strap length so the arm is comfortably supported at a 90° angle or slightly elevated for comfort.
- Make sure the wrist is fully supported and the elbow is not hanging outside the sling.
- The sling is suitable for both the left and right arm.

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