

Manual: Walking cane Quadro

DE: Gehstock Quadro	LV: Pastaigu nūja – Quadro
DK: Spadserestok – Quadro	NL: Wandelstok Quadro
EE: Jalutuskepp – Quadro	NO: Spaserstokk – Quadro
EN: Walking cane - Quadro	PT: Bengala - Quadro
ES: Bastón Quadro	RU: Трость для ходьбы четырехопорная
FI: Kävelykeppi – Quadro	SE: Promenadkäpp – Quadro
FR: Canne de marche Quadro	SI: Quadro sprehajalna palica
IT: Bastone Quadro	
LT: Vaikščiojimo lazda keturkoju stovu	

**Be sure to read all information and directions before use!
Keep this manual for future use.**

SAFETY SUMMARY

These instructions must be followed to ensure the safe operation of the Quadro cane.

WARNINGS

Do **NOT** install or use this equipment without first reading and understanding these instructions.

Examine the rubber tip for wear, tear and cracks. Replace the rubber tip immediately if any of these conditions exist.

An individual weighing up to 113kg can use the Quadro cane for ambulatory assistance. Walking canes are intended to provide support, increased stability, and assistance. The canes are **NOT** intended to support an individual's entire weight.

Make sure the snap button fully protrudes through the cane's adjustment hole. This ensures that the cane is securely locked in place.

Before using the cane, ensure that it is **securely locked** in place or a fall could result, causing bodily injury and/or damage to the cane.

1. ADJUSTING

- Loosen the lock knob.
- Depress the snap button and pull handle until snap button locks into the next height adjustment hole of the cane.
NOTE: An audible "CLICK" will be heard when snap button securely protrudes through the adjustment hole.
- When the cane handle is firmly positioned at desired height and snap button protrudes through height adjustment hole. Turn the lock knob clockwise to secure.

2. PROPER CANE HEIGHT

For a more comfortable fit, ensure shoe height is the same as shoe style worn most frequently when using cane.

- Stand as straight as possible on a firm, flat surface, and let your arm hang down naturally by your side.
- Adjust the cane so that the top of the handle touches the inside of the wrist where it meets the palm.

3. MAINTENANCE & CARE

- Ensure snap button releases and re-engages properly.
- Inspect cane tips for wear and damage.
- Replace any broken, damaged or worn items immediately.

Weight capacity 113 KG

Always consult your physician or therapist to determine proper adjustments and usage.

For more languages and product info please check www.vitality.com

Manual: Walking cane Quadro

DE: Gehstock Quadro	LV: Pastaigu nūja – Quadro
DK: Spadserestok – Quadro	NL: Wandelstok Quadro
EE: Jalutuskepp – Quadro	NO: Spaserstokk – Quadro
EN: Walking cane - Quadro	PT: Bengala - Quadro
ES: Bastón Quadro	RU: Трость для ходьбы четырехопорная
FI: Kävelykeppi – Quadro	SE: Promenadkäpp – Quadro
FR: Canne de marche Quadro	SI: Quadro sprehajalna palica
IT: Bastone Quadro	
LT: Vaikščiojimo lazda keturkoju stovu	

**Be sure to read all information and directions before use!
Keep this manual for future use.**

SAFETY SUMMARY

These instructions must be followed to ensure the safe operation of the Quadro cane.

WARNINGS

Do **NOT** install or use this equipment without first reading and understanding these instructions.

Examine the rubber tip for wear, tear and cracks. Replace the rubber tip immediately if any of these conditions exist.

An individual weighing up to 113kg can use the Quadro cane for ambulatory assistance. Walking canes are intended to provide support, increased stability, and assistance. The canes are **NOT** intended to support an individual's entire weight.

Make sure the snap button fully protrudes through the cane's adjustment hole. This ensures that the cane is securely locked in place.

Before using the cane, ensure that it is **securely locked** in place or a fall could result, causing bodily injury and/or damage to the cane.

1. ADJUSTING

- Loosen the lock knob.
- Depress the snap button and pull handle until snap button locks into the next height adjustment hole of the cane.
NOTE: An audible "CLICK" will be heard when snap button securely protrudes through the adjustment hole.
- When the cane handle is firmly positioned at desired height and snap button protrudes through height adjustment hole. Turn the lock knob clockwise to secure.

2. PROPER CANE HEIGHT

For a more comfortable fit, ensure shoe height is the same as shoe style worn most frequently when using cane.

- Stand as straight as possible on a firm, flat surface, and let your arm hang down naturally by your side.
- Adjust the cane so that the top of the handle touches the inside of the wrist where it meets the palm.

3. MAINTENANCE & CARE

- Ensure snap button releases and re-engages properly.
- Inspect cane tips for wear and damage.
- Replace any broken, damaged or worn items immediately.

Weight capacity 113 KG

Always consult your physician or therapist to determine proper adjustments and usage.

For more languages and product info please check www.vitality.com



CE Date of issue: 22-09-2022
Version: 1.0

CE Date of issue: 22-09-2022
Version: 1.0

MIXED.Industries B.V. Lovense Kanaaldijk 45-4, 5013 BJ Tilburg, The Netherlands	MIXED.Industries B.V. Lovense Kanaaldijk 45-4, 5013 BJ Tilburg, Niederlande	MIXED.Industries B.V. Lovense Kanaaldijk 45-4, 5013 BJ Tilburg, Pays-Bas	MIXED.Industries B.V. Lovense Kanaaldijk 45-4, 5013 BJ Tilburg, Países Bajos	MIXED.Industries B.V. Lovense Kanaaldijk 45-4, 5013 BJ Tilburg, Paesi Bassi	MIXED.Industries B.V. Lovense Kanaaldijk 45-4, 5013 BJ Tilburg, Nederland
Do you have any questions? Call us +31 (0) 13 544 61 24 You can reach us on weekdays from 8:30 until 17:00 (GMT + 1)	Haben Sie Fragen? Rufen Sie uns an unter +31 (0) 13 544 61 24 Sie erreichen uns unter der Woche von 8:30 Uhr bis 17:00 Uhr (UTC+1)	Vous avez des questions? Appelez-nous au +31 (0) 13 544 61 24 Nous sommes joignables tous les jours ouvrables de 8 h 30 à 17 h 00 (GMT + 1)	¿Tiene alguna pregunta? LLámenos al número de teléfono +31 (0) 13 544 61 24 Puede contactarnos en días laborables de 8:30 a 17:00 (GMT + 1)	Hai delle domande? Chiamaci +31 (0) 13 544 61 24 Puoi contattarci nei giorni feriali dalle 8:30 alle 17:00 (GMT + 1)	Heeft u nog vragen? Bel ons +31 (0) 13 544 61 24 U kunt ons bereiken op werkdagen van 8:30 tot 17:00 (GMT + 1)
For more information, visit vitality.com	Weitere Informationen finden Sie auf vitality.com	Pour de plus amples informations, consultez vitality.com	Para obtener más información, visite vitality.com	Per ulteriori informazioni, visita vitality.com	Ga naar vitality.com voor meer informatie
Made in Taiwan	Hergestellt in Taiwan	Fabriqué en Taiwan	Fabricado en Taiwan	Prodotto in Taiwan	Gemaakt in Taiwan

MIXED.Industries B.V. Lovense Kanaaldijk 45-4, 5013 BJ Tilburg, The Netherlands	MIXED.Industries B.V. Lovense Kanaaldijk 45-4, 5013 BJ Tilburg, Niederlande	MIXED.Industries B.V. Lovense Kanaaldijk 45-4, 5013 BJ Tilburg, Pays-Bas	MIXED.Industries B.V. Lovense Kanaaldijk 45-4, 5013 BJ Tilburg, Países Bajos	MIXED.Industries B.V. Lovense Kanaaldijk 45-4, 5013 BJ Tilburg, Paesi Bassi	MIXED.Industries B.V. Lovense Kanaaldijk 45-4, 5013 BJ Tilburg, Nederland
Do you have any questions? Call us +31 (0) 13 544 61 24 You can reach us on weekdays from 8:30 until 17:00 (GMT + 1)	Haben Sie Fragen? Rufen Sie uns an unter +31 (0) 13 544 61 24 Sie erreichen uns unter der Woche von 8:30 Uhr bis 17:00 Uhr (UTC+1)	Vous avez des questions? Appelez-nous au +31 (0) 13 544 61 24 Nous sommes joignables tous les jours ouvrables de 8 h 30 à 17 h 00 (GMT + 1)	¿Tiene alguna pregunta? LLámenos al número de teléfono +31 (0) 13 544 61 24 Puede contactarnos en días laborables de 8:30 a 17:00 (GMT + 1)	Hai delle domande? Chiamaci +31 (0) 13 544 61 24 Puoi contattarci nei giorni feriali dalle 8:30 alle 17:00 (GMT + 1)	Heeft u nog vragen? Bel ons +31 (0) 13 544 61 24 U kunt ons bereiken op werkdagen van 8:30 tot 17:00 (GMT + 1)
For more information, visit vitality.com	Weitere Informationen finden Sie auf vitality.com	Pour de plus amples informations, consultez vitality.com	Para obtener más información, visite vitality.com	Per ulteriori informazioni, visita vitality.com	Ga naar vitality.com voor meer informatie
Made in Taiwan	Hergestellt in Taiwan	Fabriqué en Taiwan	Fabricado en Taiwan	Prodotto in Taiwan	Gemaakt in Taiwan